MiroCam®

MiroCam®

Smallest but Highest Resolution

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Attach sensor pads according to the numbers on the data cables.

- 1 Locate the center between the umbilical and the xiphoid process.
From this center point, attach the sensor to the distal point on the right flank of the patient's body.
- 2 Place the sensor pad on the xiphoid process line, directly below the right clavicle.
- 3 Place the sensor pad on the xiphoid process line, directly below the left clavicle.
- 4 Locate the center between the umbilical and the xiphoid process.
From this center point, attach the sensor to the distal point on the left flank of the patient's body.
- 5 Place the sensor pad on the right pelvic line, 2 cm behind sensor #1.
- 6 Place the sensor on the right inguinal line, 2 cm to the outside of sensor #2.
- 7 Place the sensor on the left inguinal line, 2 cm to the outside of sensor #3.
- 8 Place the sensor pad on the left pelvic line, 2 cm behind sensor #4.
- R Place the sensor directly below the right clavicle.

The numbers of the sensor pads must be matched to the relevant area.

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Step 2 Step-by-Step Patient Preparation

1 Align the Data belt

- Open the packaging of the Data Belt
- The Data Belt sensors should be placed on the abdomen of the patient
- The Receiver connector should be on the left side of the patient's abdomen

2 Wear the Data belt

- Align the umbilical button with the stitched area between the sensor pads
- Pull the velcro around the belt for a slight fit
- Do not attached the Data belt above clothing

3 Attach receiver

- Turn on the MicroCam® receiver and attach the Data Belt connector

4 Remove the Capsule

- Prior to handling the capsule, the patient and nurse(s) should wash hands.
- Grasp the plastic casing covering and separate the capsule from the case.
- The capsule package should be not damaged.
- When the capsule is separated from the case, it should be blinked.
- Patients should grip the gold section of the capsule between the thumb and the forefinger of the both hands.

- Never touch the dome of the capsule.
- Do NOT use a capsule if the package is damaged.

5 Check Receiver Operation

- Check the battery state. The battery indicator should be green. If it is orange, fully charge the battery.
- Check the initialization state. – The initialization indicator should be lit green. If it is orange, initialize the receiver.
- Connect the data cable to the MicroCam® receiver.
- SIG indicator will turn green after the patient holds gold bands of the capsule with hands (see step 4).

If the battery indicator on the Receiver shows that the battery is not fully charged, do not start the endoscopy procedure. The battery may be depleted during the procedure and images may not be saved properly.

6 Ingest the Capsule

- After confirming the SIG indicator is green, the patient should swallow the capsule with a small amount of water.
- Tell Patient not to bite the capsule before ingesting
- The patient should drink lots of water.

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Copyright 2007 IntroMedic